




# VANISHpoint™


## Syringe

### ONE STEP


Standard Draw Procedure

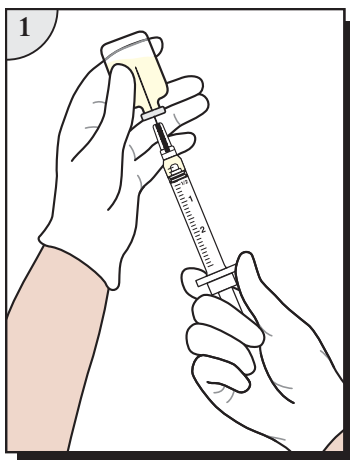

One-Handed, Automatic Retraction

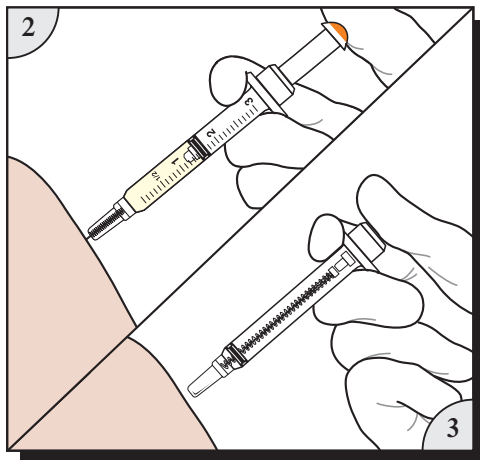

Non-Reusable

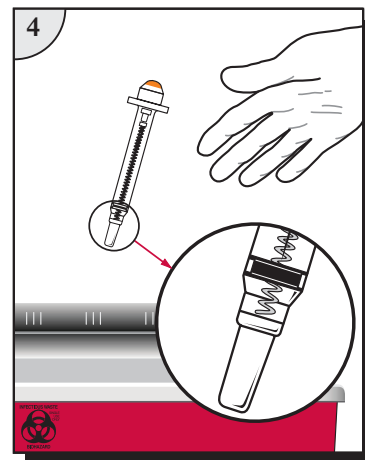

### Precautions:

- Single use only.
- Contents are sterile, non-toxic, and non-pyrogenic. Do not use if product or package is damaged.
- Contains no natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- For applications where full contents are not administered, expel remaining contents and activate needle retraction.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

### Product Usage Information:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. For injection into patients, continue depressing plunger to activate automatic needle retraction **while needle is still in patient**.  
For injection into IV ports, continue depressing plunger to activate automatic needle retraction and **immediately remove needle from port. Full dose is administered only when needle retraction is activated.**
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable.
4. Dispose of VanishPoint® syringe in an appropriate sharps container per protocol of institution.